SMARTACT evo

CE
0123

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- (F) Upper ring nut
- (G) Inertial system
- (H) Curved body
- (I) Head ring nut
- (J) Inertial system locking wrench

A2

- (L) Single-use tack insertion device
- (M) Single-use striker
- (N) Titanium tack with pin holder and protective cap

Intended use

SMARTACT^{evo} is a pneumatic system that allows positioning tacks to fixate and stabilize the membrane in bone regeneration in oral surgery. It can be used to stabilize both resorbable and non-resorbable membranes. SMARTACT^{evo} is equipped with a pneumatic pedal that allows imparting a force (adjustable depending on the type of bone), which instantly inserts the tack, thus fixating the membrane firmly to the bone. The extremely compact handpiece is designed with a tapered shape and curved tip so that even the most inaccessible areas of the mouth can easily be reached. SMARTACT^{evo} is made of material free of hazardous substances and resistant to corrosion.

- SMARTACT^{evo} is supplied non-sterile. It must be washed and disinfected and some components sterilized before each use.
- The SMARTACT^{evo} device must be used exclusively by skilled medical staff.
- The surgeon must assess whether the patient is suitable for bone regeneration operations and the appropriate surgical procedure.
- Never use SMARTACT^{evo} without ensuring that all its components have been assembled correctly (see assembly instructions).
- Assemble the handpiece first and then connect the instrument to the pedal to prevent accidental activation. The medical staff and the patient must wear protective goggles. Carefully follow the assembly sequence (see assembly instructions).
- Never operate SMARTACT^{evo} without having inserted the single-use tack insertion device and the tack.
- Make sure that you correctly insert the single-use tack insertion device and the single-use striker (see Photos 4 and 5) and do not detach the titanium tack from the pin holder before positioning (Photo 19) to prevent the risk of accidental ingestion by the patient.
- When the instrument is not in use, adjust it to minimum force for safety.
- Do not leave the instrument connected to the compressed air (even at minimum force) once you have completed the operation.
- Before use, check that the instrument package is intact.
- Do not use the product if the package is damaged.
- Always wear sterile gloves when handling the SMARTACT^{evo} device and follow strict procedures in order to guarantee sterility.
- After use, discard in special medical waste containers in compliance with the regulations in force.
- Meta cannot be held responsible for improper use of the product.

**How to assemble the instrument before use**

Strictly follow the assembly sequence; failed, partial or incorrect assembly of a component may result in:

- failed or improper functioning;
- damage to the instrument.

The assembly sequence is shown in Scheme 1.

When you have assembled the instrument, it is recommended to cover the cable and handpiece with appropriate single-use protective sheaths to facilitate subsequent washing and sterilization of the instrument.

How to assemble the SMARTACT^{evo} Pin tacks on the instrument

Strictly follow the sequence to load the device. Failed, partial or incorrect assembly of a component may result in:

- risk of accidental ingestion of the tack, tack insertion device or striker by

the patient;

- failed or improper functioning;
- damage to the instrument.

The assembly sequence is shown in Scheme 2.

It is strongly inadvisable to use tacks from other manufacturers. Meta cannot be held responsible for this improper use of the instrument, which may put the patient at risk and seriously damage the device.

How to adjust and use the instrument

After positioning the tack on the head of the handpiece as explained in Scheme 2, position the instrument (in line with the receiving plane) on the insertion area and operate the pedal (fully depress the pedal and release it). If necessary, depress the pedal again until the tack has completely been inserted. Based on the clinical case and with reference to the bone density and crest anatomy (thickness, slope, etc.), the surgeon should decide how many times the pedal needs to be depressed.

The instrument is fitted with an adjustment ring nut that allows the user to select the suitable insertion force by turning the ring nut from minimum to maximum.

It is recommended to adjust to minimum for the lateral sinus floor areas.

It is recommended to adjust to intermediate for the upper jaw areas with variable cortical bone thickness and bone density Type II or III.

It is recommended to adjust to maximum for the lower jaw areas with cortical bone density type I.

Before adjusting SMARTACT^{evo}, it is recommended to perforate the cortex in order to test the bone consistency.

How to disassemble the instrument after use

After using the instrument, disassemble it following the assembly instructions shown in Scheme 1 in reverse order. Thoroughly wash and clean the instrument before putting it away.

How to wash and sterilize the instrument

The device is supplied non-sterile. The pedal, the cable and the handpiece body (components A, B, C, D, E - see Photo A1) must be washed and disinfected before each use.

NB: always close the compressed air intake and disconnect the cable from the connector before disassembling the instrument.

The components of the upper part of the handpiece: upper ring nut, curved body, head ring nut, inertial system and locking wrench of the inertial system (components F, G, H, I, J - see Photo A1) must be washed, decontaminated and sterilized before each use.

Wash with appropriate germicidal detergents immediately after the surgical procedure. Use suitable detergents; do not use hydrogen peroxide (H₂O₂) or chlorine-based detergents.

The components that must be sterilized (F, G, H, I, J) after washing must be placed in an appropriate bag for subsequent sterilization.

The components must be sterilized in a steam autoclave at 121°C for 30 minutes, always using validated sterilization cycles in compliance with the regulations in force. Sterility is guaranteed if the bag is intact. Always check the bag before use.

Maintenance

Routine maintenance consists of washing and cleaning the instrument according to the above mentioned procedures.

Connecting the instrument to the compressed air

The instrument must be connected to the compressed air system.

If the pressure is:

- less than 4 bar: the operating force will be insufficient;
- more than 4 bar: the higher pressure will affect the operating speed of the instrument; the force imparted on the tack is in any case independent of the operating pressure.

The recommended operating pressure is between 4 and 6 bar.

Commercial warranty on the instrument

The commercial warranty offered by Meta is a guarantee of good functioning against defects arising as a result of long-term use.

The SMARTACT ^{evo} device is covered by warranty in accordance with the law for 12 months from the date of sale stated on the invoice.

The warranty does not cover improper use of the device. Improper use includes the following: repairs and maintenance performed by persons not authorised by the manufacturer, defects or damage caused by falling, breaking, lightning strikes or liquid penetration, defects or damage caused by mechanical, chemical or thermal effects, devices fitted with components or accessories not authorised by the manufacturer. In order to exercise their rights, customers must report any conformity defect within two months from discovery of such defect. In the event of replacement of a product or

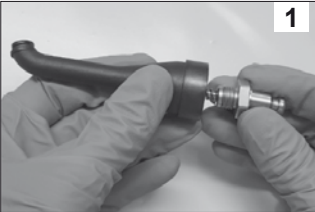
component, the products or individual components returned for replacement become the property of the manufacturer. A service provided under warranty does not extend the original warranty period. Therefore, in the event of replacement of a product or component, the part that is supplied in replacement of a defective part shall not be warranted longer than the expiration of the original warranty period of the defective part. All other rights are excluded.

These instructions for use as well as further information on the instrument and the assembly and disassembly sequences of SMARTACT ^{evo} are available on the website www.metahosp.com

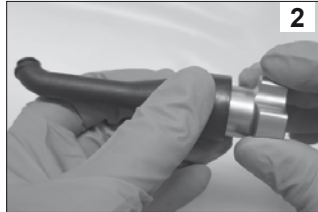
Only by reading this documentation can you avoid errors and ensure proper functioning of our product.

SCHEME 1 - INSTRUMENT ASSEMBLY

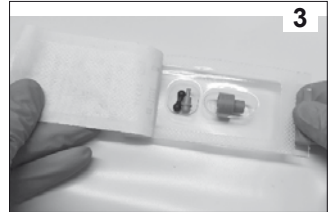
Assembling the handpiece

1 Insert the inertial system (G) into the curved body (H)

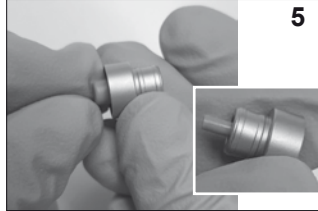
2 Screw it fully into place using the dedicated locking wrench provided (J). NB: Firmly tighten into place

3 Open the sterile SMARTACT ^{evo} Pin blister pack containing the single-use pins and components.

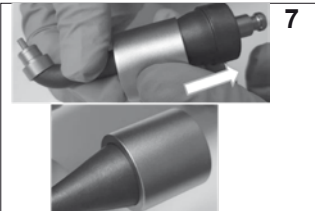
4 Take the single-use striker (M) from the blister pack and insert it into the curved body (H)

5 Take the single-use tack insertion device (L) from the blister pack and insert it into the head ring nut (I)

6 Screw the head ring nut into place (with the single-use tack insertion system as shown in Photo 5) on the head of the curved body (H) containing the single-use striker (M)

7 Insert the curved body (H) into place in the upper ring nut (F)

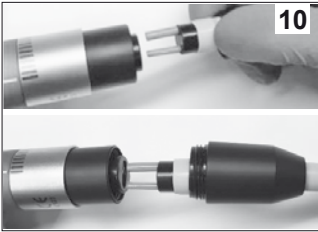
8 Screw the curved body with ring nut (Photo 7) onto the upper side of the central body of the handpiece (E) tightening it into place.

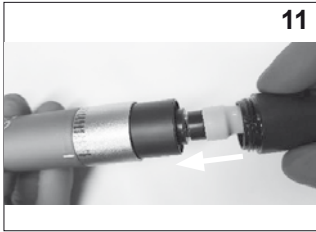
9 Assembled handpiece

Pay the utmost attention and make sure that you have correctly engaged the thread and firmly screwed the ring nuts into place.

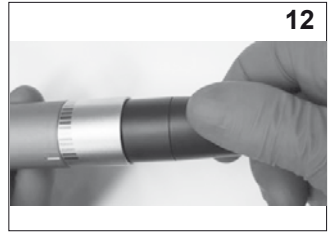
Connecting the handpiece to the pedal

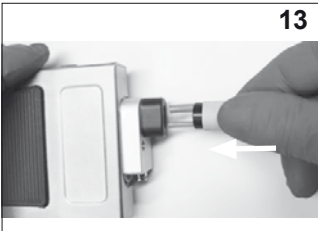
Connect the end (the one with the cable cover) of the connection cable (B) to the handpiece**

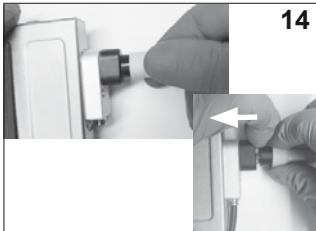
Push the tubes leading from the connection cable fully into place in the handpiece couplings

Screw the cable cover on the connection cable (B) to the base of the handpiece

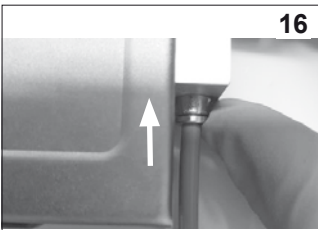
Connect the other end of the connection cable (B) to the pedal (A) in the same way as described in Photos 10 and 11.

The tubes can be disconnected from the pedal or handpiece by pushing the coupling block downward and pulling the cable

Connect the end of the compressed air cable (C) to the side coupling of the pedal (A) pushing it fully into place

The compressed air cable can be disconnected from the pedal by pushing the coupling downward and pulling the cable

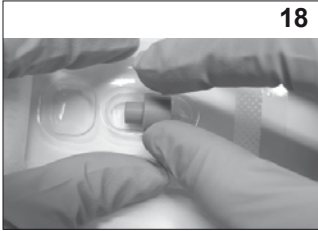
Set the operating pressure to not less than 4 bar (maximum permitted 6 bar)

Connect the end of the compressed air cable (C) to the compressed air supply

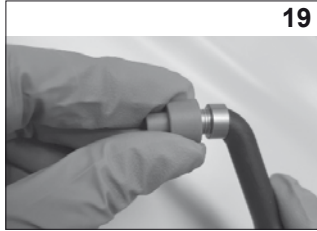
** Take care to insert the smaller diameter connection fitting into the smaller diameter coupling and the larger diameter fitting into the larger diameter coupling.

SCHEME 2 - SMARTACT EVO TACK ASSEMBLY ON THE INSTRUMENT

18

Take the titanium tack in its protective cap (N) from the blister pack

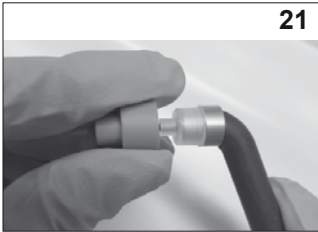
19

Insert the handpiece head (Photo 6) into the cap (N) and push it until it locks into place

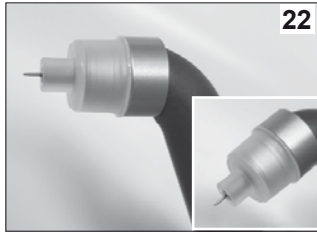
20

You should hear a click as the pin holder properly fits into place on the handpiece head

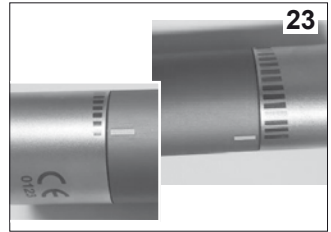
21

Remove the cap that has detached from the pin holder

22

Pin holder (containing the titanium tack) correctly positioned

23

Turn the lower ring nut to adjust to the suitable insertion force

24

SMARTACT evo system ready for use

25

STAY AWAY FROM THE SURGICAL SITE
Fully depress the pedal (A) and release it to allow the pin holder to detach from the curved body head (H)

26

WARNING: pin holder expulsion is automatically activated when the pedal is depressed. During this operation, always point the tip of SMARTACT evo towards a waste collection container

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C.G.M. S.p.A. DIVISIONE MEDICALE META